

Reminder

Please avoid sugary drinks, tea, coffee wine, and eating when wearing aligners or retainers to ensure the best results and maintain your oral hygiene.

Tips

Rinse the rejuvenated appliances in cold tap water and gently brush to remove any tablet residue from the surface. The residue is easily washed away with a gentle rub, leaving your appliances feeling fresh and clean.

Note: Do NOT brush under HOT tap water, which might cause damage to the appliance.



For Patients How to Clean Your Aligner/Retainer?

Who have 4D Clean Kit



STEP 1

- Place 4D Aligners™ side by side, teeth cavities face-up, onto the filter rack, into the cleaner.



STEP 2

- Add tap water into cleaner, submerging appliances, below the max water level.
- If you want